


SyrVet™ Oral Fluidfeeder



The flexible plastic esophageal probe directs fluids and vital nutrients directly to the stomach!

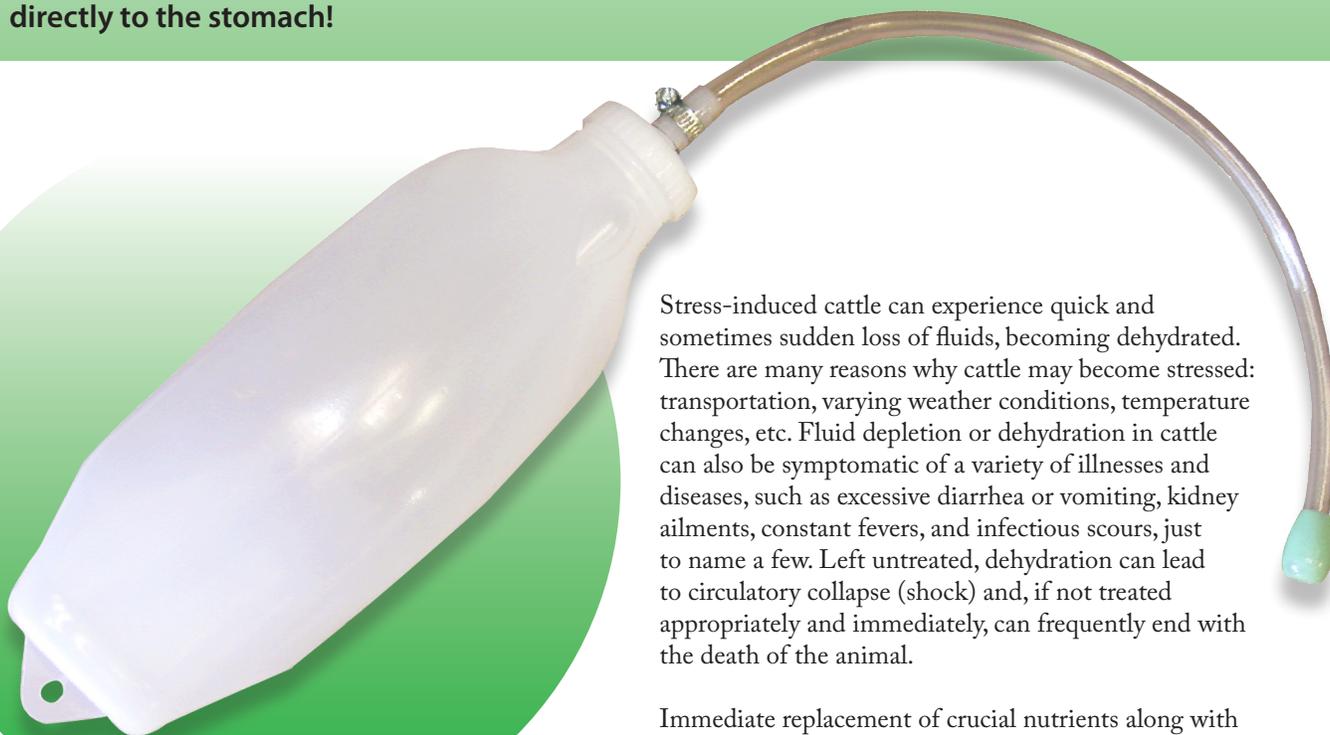

Stress-induced cattle can experience quick and sometimes sudden loss of fluids, becoming dehydrated. There are many reasons why cattle may become stressed: transportation, varying weather conditions, temperature changes, etc. Fluid depletion or dehydration in cattle can also be symptomatic of a variety of illnesses and diseases, such as excessive diarrhea or vomiting, kidney ailments, constant fevers, and infectious scours, just to name a few. Left untreated, dehydration can lead to circulatory collapse (shock) and, if not treated appropriately and immediately, can frequently end with the death of the animal.

Immediate replacement of crucial nutrients along with high quality drinking water is vital for rapid recovery. Drugs alone won't help. The most recommended procedure for rehydration is to administer the right mix of ingredients to the animal. One of the most efficient veterinary procedures for doing this is through oral fluid feeding.

Directions For Use

1. Pour drench feed product into the container, following feed product manufacturer's and/or veterinarian's suggested amount.
2. Add water according to directions on drench feed product.
3. Screw on the cap and tubing assembly to container.
4. Fold the tubing against the lid slightly to avoid losing liquid while inserting tubing into the animal.
5. Insert the tube, staying on top of the tongue, into the left side of the calf's mouth and push it about halfway down the throat. The tube should automatically be directed to the esophageal opening.
6. Holding the container at the bottom, begin squeezing, forcing the liquid out, repeating until container is completely drained.

For veterinary use only. This product is not intended for, or to be used by, children. As with all procedures, please consult your veterinarian for proper instructions prior to use.



SyrVet, Inc.
PO Box 490 | 955 SE Olson Dr.
Waukee, IA | 50263 U.S.A.
Toll: 800.727.5203
Ph: 515.987.5554
Fax: 515.987.5553
customerservice@syrvet.com
www.syrvet.com