

THE 3 OVIJECTORS

DESIGN Your Kaycee Ovijector gives comfort and speed of handling with accuracy, easy maintenance and long life. Quality materials are used throughout and each instrument is factory-tuned using digital calibration.

Cat 10030 – OVIJECTOR Mk 1


0.1 to 1.0ml in steps of 0.1ml Luer Lock. Incl. 12 reusable hypo 22g x 13mm.

Cat 10042 – OVIJECTOR Mk 2

0.5 to 2.0ml in steps of 0.25ml Luer Lock. Incl. 12 reusable hypo 18g x 13mm.

Cat 10054 – OVIJECTOR Mk 3

1.0 to 5.0ml in steps of 0.5ml Luer Lock. Incl. 12 reusable hypo 18g x 13mm.

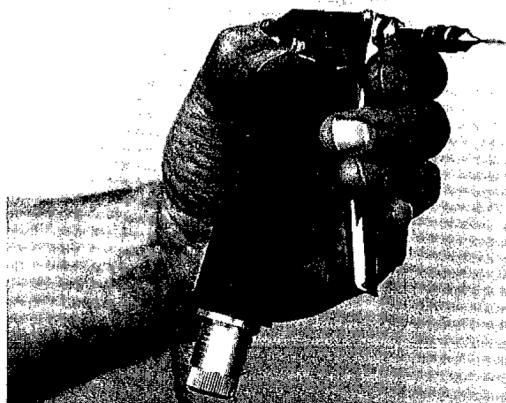


INSTRUCTIONS FOR USING YOUR KAYCEE VACCINATOR

STERILISATION

1. Remove PVC grip with anticlockwise quarter turn and set aside. **DO NOT STERILISE GRIP.**
2. Pump to fill syringe with previously-boiled or distilled water, wrap in clean white cloth with vaccine feed tube and any needles to be used, and immerse. **BOIL 10 MINUTES.**
3. Pump out water and refit handle.

CONNECT VACCINE - Fit PVC tube to rear connector and short feed needle. Use an air needle in vaccine bottle to relieve vacuum. Fit injection needle to Luer Lock, turning to fix. Operate trigger until vaccine flows.

SETTING THE DOSE - Set the dose by twisting the dose knob. It will click into precise dose settings each complete turn. Read the dose setting against the extreme end of the Pistol Grip.

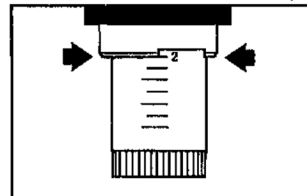

AFTER USE - clean thoroughly; immediately after use pump through plenty of hot soapy water, followed by clean water. Then remove piston after detaching from cross-pegs of side arms, smear 'O' ring (piston seal) with liquid paraffin or castor oil and replace. *Do not interchange pistons between syringes* - they are individually calibrated for accuracy.