



1-800-433-6340 • www.qcsupply.com

Sterimatic® Vaccination Pack

Sterimatics® allow producers to administer safer, cleaner injections. The attachment acts as a needle guard to prevent accidental self-injection, while also preventing cross infection and the spread of disease. The white Stericap® on the end of the guard is filled with a disinfectant agent consisting of glutaraldehyde (2.5%) and Bardac 22 (5%). A sponge within the Stericap® is soaked with the disinfectant and wipes the needle when it passes in and out. Each Stericap® will last for 100 injections. The Stericaps® are replaceable and the needle can be changed quickly. For correct dosage and administration; a 1" needle must be used with the ½" Sterimatic® and a 1½" needle must be used with the 1" Sterimatic®.



141255- ½" Sterimatic®
with Stericap® and 1" needle

Two Sizes Available:** ½" and 1".

****For correct dosage and administration** a 1" needle must be used with the ½" Sterimatic® and a 1½" needle must be used with the 1" Sterimatic®.



141255- 1" Sterimatic®
with Stericap® and 1½" needle

Advantages:

- Automatically cleans with every injection
- Reduces infection and abscessing
- Reduces disease transmission
- 100 injections/Stericap® (within 3 days); 500 injections for poultry

Benefits to User:

- Safety - due to a reduction in needle stick and scratch injuries
- Financial - due to healthier livestock; less infections, abscesses and lower mortality rates
- Confidence - due to better and safer injection methods

Benefits to Livestock:

- Reduced cross infection of disease between livestock
- Reduced bacterial infection at the injection site (reduced abscessing)
- Healthier livestock

How it Works: After attaching the sleeve and Stericap® to the syringe you inject as normal. As the Stericap® is pushed against the skin, the sleeve automatically retracts. The needle is cleaned as it passes through into the animal. The dose can be delivered as normal. Then, as the needle is withdrawn, the sleeve automatically extends and the needle is cleaned for a second time and then covered and protected. Use each Stericap® for up to 100 injections within 3 days of opening.

141255.....	½" Sterimatic® with Stericap® and 1" needle.....	\$6.95
141255.....	1" Sterimatic® with Stericap® and 1½" needle.....	6.95